


III. AnyRidge Prosthetic Kit

Ref.C
KANPK3000

A kit with all kinds of driver that are needed for prosthetics.

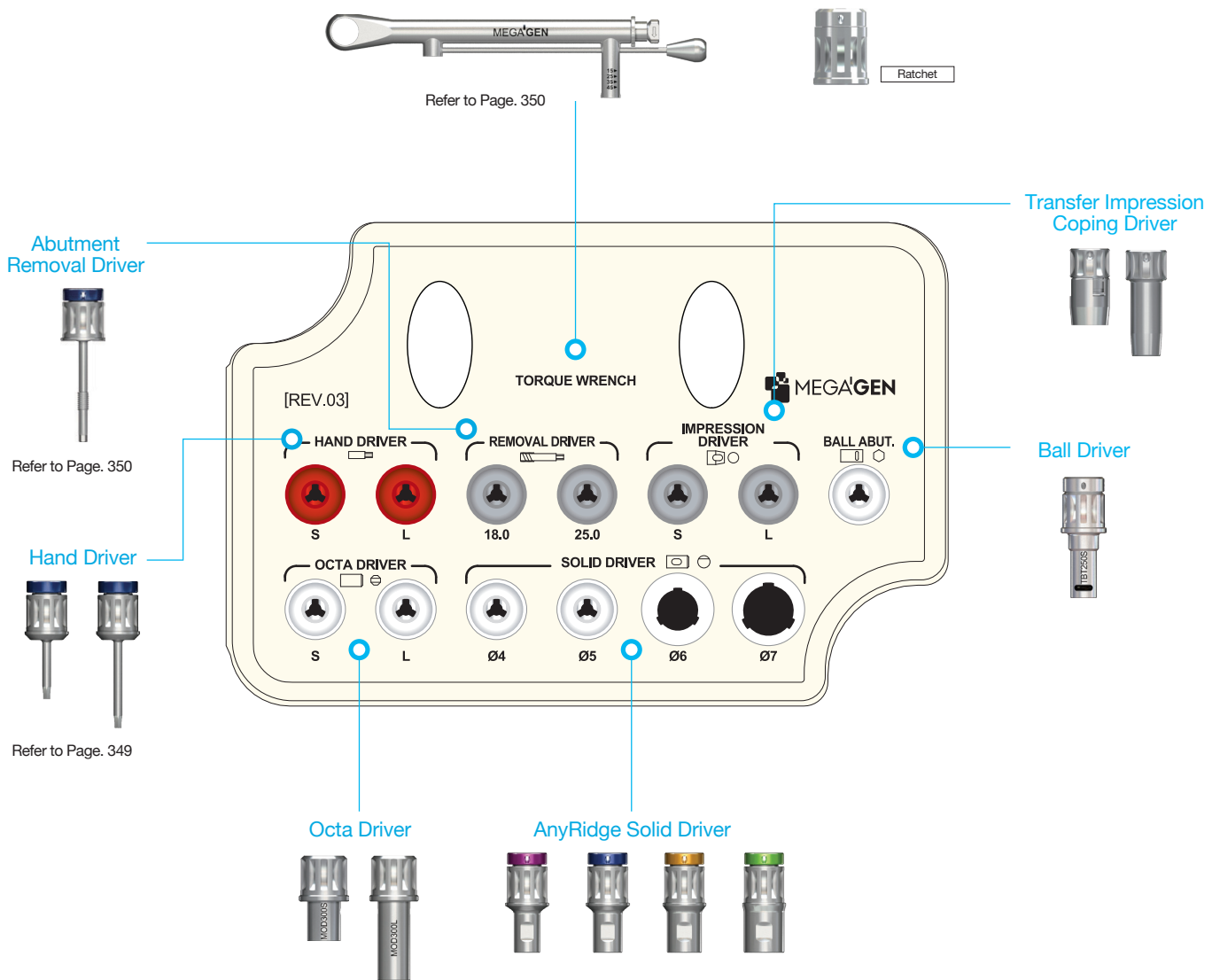


➔ Prosthetic Kit Components

Solid Driver

- For the delivery of Solid Abutments.
- Color coded for different profile diameters. (Ø4-magenta, Ø5-blue, Ø6-yellow, Ø7-green)
- Two different heights. (8.5 / 13.5mm)
- Directly connectable to Torque Wrench.

Solid Abutment Profile Diameter	Length(mm)	Ref.C
Ø4	8.5	TANSDS400
	13.5	*TANSDL400
Ø5	8.5	TANSDS500
	13.5	*TANSDL500
Ø6	8.5	TANSDS600
	13.5	*TANSDL600
Ø7	8.5	TANSDS700
	13.5	*TANSDL700

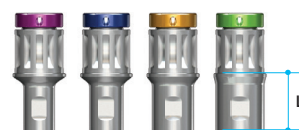

(*) Separate sales item.

Octa Driver

- For seating of the Octa Abutment into the fixture.
- Can also be connected to Torque Wrench.


Length (mm)	Ref.C
7	MOD300S
13	MOD300L



Ball Driver

- For seating of the Ball Abutment into the fixture.
- Can connect to a Handpiece, Ratchet or Torque Wrench.
- Available in long and short.

Type	Ref.c
*Handpiece Connector(Short)	TBH250S
*Handpiece Connector(Long)	TBH250L
*Ratchet Connector(Short)	TBR250S
*Ratchet Connector(Long)	TBR250L
Toque Driver(Short)	TBT250S
*Toque Driver(Long)	TBT250L

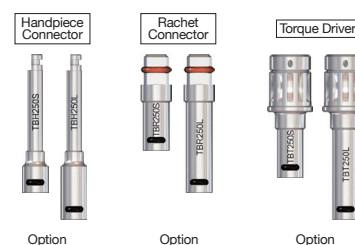


(*) Separate sales item.

Impression Coping Driver (Transfer)

- For transfer type of Impression Coping.
- Works with friction only.
- Small but powerful grip.

Type	Ref.C
For Two piece impression Coping	TCMID
For One piece impression Coping	TCMIDE

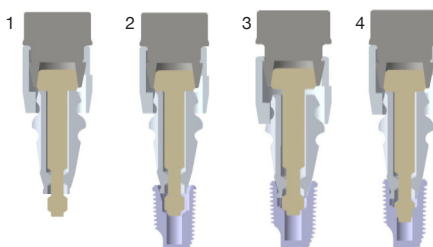

1. Connect Impression Coping and Impression Driver together
2. Adjust Connection with a Fixture by turning a Holder clockwise.
3. Push the Holder and put the Impression Coping into the Fixture.
4. Turn the Driver clockwise to ensure connection of the Impression Coping and Fixture.